FARE® – MUNDSCHIRM

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*Represented motif
as an example only.
The product is custom made
with your individual design!*

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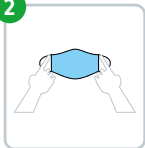
Wash your hands with soap and water or rub with a hand sanitiser before any handling of the community mask.

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Enter with the chin, then pull the mask over the nose so that the mask covers the nose, mouth and chin completely. Attach elastic bands behind the ears and adjust the size.

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Adjust nose clip with index finger and thumb.

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Check that the mask fits well and is tight. Each time the community mask is touched, the user must wash the hands with soap and water or rub with a hand sanitiser.

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Community mask

- *Reusable*
- *Upper material: 95 % polyester, 5 % elastane for a perfect fit*
- *Lining: 100 % cotton for high comfort*
- *Washable at 60° C*
- *Two-layer design with insertion possibility for a filter / cloth¹*
- *With elastic bands for fastening behind the ears, adjustable*
- *With nose clip*
- *Dimensions: approx. 18.5 x 13 cm*

*adjustable
elastic bands*

*insertion possibility for a
filter / cloth*

Manufacturer / supplier:
FARE – Guenther Fassbender GmbH
Stursberg II 12
D-42899 Remscheid
info@fare.de
www.fare.de

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Please note

- Use your mask for private use only
- Continue to comply with current hygiene regulations, in particular the current recommendations of the Robert Koch Institute (RKI, www.rki.de/en) and the Federal Centre for Health Education (BZgA, www.bzga.de/home/bzga)
- When putting on the mask, be careful not to contaminate the inside of your mask. Wash your hands thoroughly with soap before putting on the mask
- Place the mask correctly over the mouth, nose and cheeks so that it fits as tightly as possible to minimize air ingress from the side
- Before using the mask for the first time, check that the mask allows enough air to pass through to minimise interference with normal breathing
- Remove and replace a soaked mask immediately
- The outside of the used mask is potentially pathogenic. To avoid contaminating hands, avoid touching the mask if possible
- Wash your hands thoroughly after removing the mask, following the general rules of hygiene (at least 20 – 30 seconds with soap)
- As soon as your masks have a finishing (e.g. printing or embroidery), the wash labels on the mask may lose their validity
- Wash your mask after use or store it in an airtight bag. If possible, the mask should be stored in a bag only for a short period of time to prevent mould growth

What to avoid

- The community mask cannot be used for protection against chemicals
- It is not possible to decontaminate community masks by freezing. The viral or bacterial agent is retained and at 4°C it does not lose its infectious character
- Keep a safe distance of at least 1.50 m from other people when wearing the mask, as recommended by the World Health Organization
- Boiling the community masks is not recommended. There is no guarantee that the fabric can withstand the 100°C temperature of boiling water without degrading

Warning

- In order to use this community mask correctly, it is important that you follow these instructions
- If you are ill, this community mask is unsuitable. Seek advice from your doctor
- This community mask is not a medical device within the meaning of Directive 93/42/CEE or Regulation EU/2017/745 (surgical masks) nor is it personal protective equipment in the sense of Regulation EU/2016/425 (respiratory protective devices)
- Always check that the community mask is correctly fitted and covers your nose, mouth and chin. It is recommended that this community mask be worn on bare skin; beards can reduce the filtration efficiency to below the limits set out
- Community masks are not suitable for children under 3 years of age. It is recommended that children between 3 and 12 years are supervised while wearing the community mask. A community mask which hinders the user's ability to breathe when first put on is deemed unsuitable. It can take time to become comfortable with the community mask
- This community mask does not replace protective measures (regular hand-washing, physical separation, reduced contact with other people). It minimizes the projection of user's respiratory droplets saliva into the environment
- Do not use when participating in vigorous physical activity
- Stop using this product at the first signs of damage
- Do not use dry cleaning and fabric softener
- Wash your mask before wearing it for the first time. Make sure that it is dried completely